Innov8 low hospital bed...











# Designed around... safe and easy ingress and egress

The Innov8 Low bed has an exceptional low height and with ts key features, can be utilised as an everyday acute ward bed without compromising on patient outcomes.

Falls prevention is a major risk management consideration in hospitals today. Sidhil's Innov8 Low acute hospital ward bed provides a fully functional solution, with the added benefit of an extremely low platform height at just 22cm.

From this minimum level, the Innov8 Low bed rises to 71cm and incorporates a comprehensive range of electrically operated functions including auto contouring, cardiac chair function and auto regression to avoid surface pinching and patient sliding.

With both Trendelenberg and reverse Trendelenberg positioning, the Innov8 Low has a safe working load of up to 255kg (40st).

designed ridges for breathability, to simplify decontamination and to improve infection control. All beds in the Innov8 range are reliable and easy to maintain, incorporating superb ergonomics in terms of both manual handling and user comfort.

T +44 (0) 1422 233 000 F +44 (0) 1422 233 010 E sales@sidhil.com W www.sidhil.com 3

## Designed around... patient safety and infection control



#### **Low Height Range**

Exceptionally low height range of 22cm to 71cm. When lowering the bed from maximum height the bed will travel at full speed until it reaches 42cm, at which point it will slow and a short warning alarm will sound to alert the carer that the bed is now travelling lower than a standard bed. The bed can only be positioned at its lowest height with the Nurse Control Panel.

#### **Mattress Platform panels**

Specially designed smooth, wipeable surface with ridges for breathability and channels to reduce liquid pooling, to simplify decontamination and to improve infection control.

#### **Side Rails**

The Innov8 Low is supplied without side rails. Standard or high cantilever style side rails are available as a factory fitted option.

## Designed around... identified clinical needs

#### Patient Handset

Patient handset is designed to give limited control to the patient. It allows control of the backrest, knee break and mattress platform height. The mattress platform height may only be adjusted between 42cm and 71cm using the patient handset, to reduce the chance of entrapment when the patient is unsupervised.



**Backrest:** up / down



Auto contour: up / down



### **CPR Function**

Easy access manual CPR handle with CPR mechanism. Electric CPR is located on the nurse handset also.



#### **Integral Bed Extension**

Integral bed extension for the taller patient (+20cm) combines ease of use and functionality.




#### Low Height Foot End

visibility for the patient. The removable head and foot ends are also available in a number of colour options for ward

**T** +44 (0) 1422 233 000 **F** +44 (0) 1422 233 010 E sales@sidhil.com W www.sidhil.com 5

## Designed around... ease of use

Nurse Handset Functions

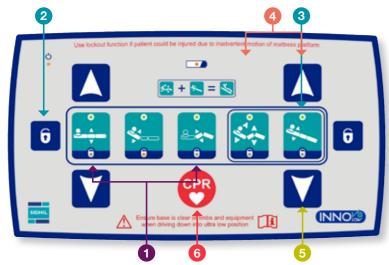

NEW Nurse Handset Holder now available as an accessory

#### 1 Platform height, backrest & leg section functions

The Innov8 iQ is a fully electric profiling bed. Using either the patient or carer handset the mattress platform height, the auto regressing backrest and leg section can easily move to ensure patient comfort and safety.

#### 2 Function lock

One or more handset functions can be locked so unintentional movement does not harm or cause discomfort to the patient.

#### 3 Tilt (Trendelenburg)

Carer can set mattress platform angle between -13° and +13°. Inbuilt software avoids entrapment and clash points with individuals and equipment.

#### 4 Cardiac chair

Cardiac chair function allows simultaneous operation of all profiling sections and tilt function, providing a comfortable seated position in bed.

#### 5 Ambidextrous operation

Carer handset can be operated with both left and right hands to suit individual preferences or switching between hands where required.

6 CPR Function

#### **Brake Bar Functions**

Brake bar/pedal ensures easy access from foot or side of the bed.


#### Triple function castors enable







castors free to roll for easy in limited spaces.

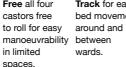



Extremely light weight bed ensures ease of manoeuvrability with minimal effort.



The Nurse Handset can now be clipped neatly away into our NEW Nurse Handset Holder

## The Innov8 Low



Factory Fitted Accessories	
Innov8 Low Bed	INNOV8/LOW/AQ/LIN

#### Bed codes with factory fitted options

Dea obacs with factory fitted options	
Innov8 Low Bed with	INNOV8/LOW/AQ/
Standard Side Rails	LINK/RSR
Innov8 Low Bed with	INNOV8/LOW/AQ/
High Side Rails	LINK/HSR
Innov8 Low Bed with Bed Pusher	INNOV8/LOW/AQ/ LINK/BP
Innov8 Low Bed with	INNOV8/LOW/AQ/
Standard Side Rails /Bed Pusher	LINK/BP/RSR
Innov8 Low Bed with High Side Rails / Bed Pusher	INNOV8/LOW/AQ/

#### **General Accessories**

Innov8 Lifting Pole	INNOV8/LP
Innov8 IV Pole with standard side rails	INNOV8/DP
Innov8 Traction - Balkan Beam	TRACT/BB/KIT
Innov8 Traction – Foot End	TRACT/FE/KIT
Oxygen Cylinder Carrier	INNOV8/RANGE/OC
Nurse Handset Holder	INNOV8/RANGE/HSH
Side Rail Pad (standard side rail)	INNOV8/LOW/SR/PAD
Side Rail Pad (high side rail)	INNOV8/LOW/HSR/PAD
Quick Release Side Rail Padding System	INNOV8/LOW/FPS
Supreme Crash Mattress	MAT/FALL/SUP
Crash Mattress (Premium)	MAT/FALL
Essentials Crash Mattress	MAT/FALL/BASIC

A full range of mattresses are available from Sidhil for the Innov8 Low

#### Dimensions 218cm (238cm Overall Length when fully extended) Overall Width 102cm with standard side rails Mattress Platform Height (minimum) 22cm 71cm Mattress Platform Height (maximum) Mattress Platform Width 90cm Mattress platform Length 200cm (220cm when fully extended) Height Overall (maximum) 120cm 70° Backrest angle 32º Knee break angle $0 - 12^{0}$ Head down tilt Foot down tilt $0 - 12^{0}$ Product weight 130kg Maximum Patient Weight 190kg (30st) Maximum Safe Working Load 255kg (40st)

## Service centres nationwide

With service centres strategically located nationwide, we guarantee an optimum response, supported by our team of field service engineers. Their experience and depth of product knowledge ensures equipment is put back into service promptly, efficiently and cost effectively.

We carry comprehensive stocks of spare parts both for the Sidhil range and for other manufacturers' products at our service centres. We also provide decontamination services to protocols as advised by a clinical scientist in infection control, staffed by trained personnel operating in accordance with MHRA DB2006 (05).

Our service centre processes are embedded within Sidhil's established quality standards to ISO 9001 levels.

For more information on our service and maintenance capabilities, contact the Service Team on: 01422 233 136.



- Glasgow
- Sunderland
- Halifax
- Warrington
- Kettering
- Orpington
- N. Ireland

#### Service & Maintenance & Rentals

Product Servicing - All Makes / Decontamination / Spares Short term solutions available as an alternative to outright purchase











Independently verified to IEC 60601-1 and IEC 60601-2-52 through a 3rd party UKAS approved test house

Sidhil Business Park Holmfield, Halifax, West Yorkshire HX2 9TN

Tel: +44 (0) 1422 233000 Fax: +44 (0) 1422 233010 Email: sales@sidhil.com Web: www.sidhil.com