

How to clean your reusable eye protection (visor/goggles)



Full face visor

Half face visor

S Lip , B White , Pamela Joannidis , E Peters v 3.0 290520




Remember to write your name in indelible ink on your eye protection





Remove gloves and apron

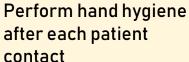




Once clinical session complete

Remove eye protection and place on trolley



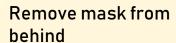



Inspection of eye protection



Perform hand hygiene



Put on clean apron and gloves



Inspect eye protection
- if not intact or there
is visible
contamination with
blood / body fluids
discard



Cleaning of eye protection



Clean eye protection with detergent wipe then an alcohol wipe. Repeat this cleaning process for the elastic/strap



Clean trolley with an alcohol wipe



Remove gloves and apron Perform hand hygiene

Clean visor to be kept in clear plastic bag